



2900 Delk Rd. Ste 700 PMB 252, Marietta, GA 30067 Tel: 770.578.5159 Fax: 770.578.4662

Scanner Settings for PracticeWorks/Kodak Dental WinOMS CS

From within the WinOMS CS software go to a patient chart, click Image button. Leave all defaults and type Test in line below. This will open the Epson Scan Software.

Settings are as follows-

(at the top, it will say Current Setting, once you create one, you click Save Setting as instructed below and it will create Setting 1, etc, so that it is now available in the drop down box for them to choose).

SETTING 1-

For X-rays, PAs, etc:

Film with Film Holder > Positive Film > 24 bit Color Descreening> Change Resolution to 96, hit Tab key on kybd > Click plus (+) sign next to Target Size, change to 50%, hit Tab key on kybd > scroll down and take the Check Mark out of Thumbnail

Click Save Setting located near top

(Should now appear in drop down list above as Setting 1)

SETTING 2-

For Insurance Cards, Drivers Licenses, EOBs, etc:

Reflective > Document > 24 bit Color Descreening > Change Resolution to 150, hit Tab key on kybd > Click plus (+) sign next to Target Size, change to 40%, hit Tab key on kybd > scroll down and make sure Check Mark is OUT of Thumbnail

Click Save Setting located near top

(Should now appear in drop down list above as Setting 2)